

Kaloidon Patch®



for the prevention and treatment of flat, hypertrophic and keloid scars.

COMPOSITION

POLYURETHANE, ALLANTOIN, ONION EXTRACT, GLYCOSAMINOGLYCANS, SODIUM HYALURONATE.

HOW A SCAR FORMS

Scars are fibrous tissues formed to repair skin lesions (pathological or traumatic) resulting from the proliferation of the dermis and epidermis. Scars usually become thinner and shrink physiologically, but if this does not occur, an enlarged scar (hypertrophic or keloid scar) will remain. LFM recommends the Kaloidon Patch® to prevent this unfortunate incidence from occurring and to improve the appearance of existing scars.

INDICATIONS

Kaloidon Patch® is a very thin, transparent bandage based on polyurethane (a material that is very compatible with the skin and also suitable for hypersensitive skin types), designed to **prevent the onset of hypertrophic and keloid scars or to treat any kind of existing hypertrophic and keloid scars (e.g. from surgical procedures such as caesarean sections, injuries, etc.)**. Kaloidon Patch® operates through a dual mechanism that combines the mechanical-compressive action of the patch on the scar whether it is hypertrophic or not, together with the functional action of ingredients such as allantoin, glycosaminoglycans and hyaluronic acid that keep the skin hydrated and elastic, thereby promoting the rapid and overall softening, **flattening and reduction of the scar**. The formulation is complemented by the anti-oxidant action of onion extract.

Kaloidon Patch® can be applied to newly formed scars immediately after removal of the stitches, thus preventing the formation of scar hyperplasia.

If applied on scars that have already formed (hypertrophic and keloid), Kaloidon Patch® can significantly improve their aesthetic appearance and function by softening the scar, and reducing and smoothing its thickness.


For these very reasons, Kaloidon Patch® can be used on both recent scars to prevent hypertrophy as well as existing hypertrophic and keloid scars.

HOW TO USE KALOIDON PATCH®


CLEANSING

Cleanse the affected area with a mild and preferably paraben- and fragrance-free detergent, and pat dry.

PATCH APPLICATION - To apply the patch, peel back one of the two protective paper parts starting from a corner and carefully place the adhesive part of the patch onto the affected area, then gently remove the second protective paper in the same way while adhering the patch to the epidermis (see drawings below).

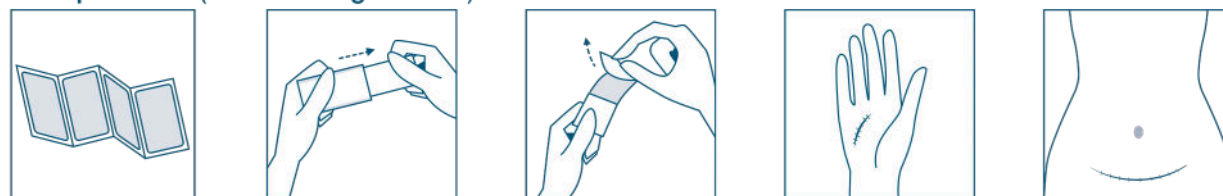

Make sure that the skin is clean and dry and that the scar is completely covered by the patch. If it is not big enough to cover the entire affected area, apply another patch nearby (at the side). Conversely, for smaller scars the patch can be cut to the desired size before removing the protective paper as shown in the drawing below.

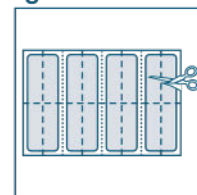

Kaloidon Patch® can be used on any part of the body, including the face. The patch should be replaced daily.

PATCH REMOVAL

Using a sponge (or a cotton swab) soaked in lukewarm water, dab the skin around the patch, then lift a flap of the patch, continue wetting the affected area and slowly remove it.



DURATION OF TREATMENT

The Kaloidon Patch® should be applied and kept in contact with the scar for a minimum of 12 hours a day.

For newly formed scars, apply a fresh patch once daily for a minimum of 4 consecutive weeks, in accordance with the doctor's recommendations. For older scars, apply a fresh patch once daily for a minimum of 8 consecutive weeks, in accordance with the doctor's recommendations. Initial results will be visible 3-4 weeks after the start of treatment. For stubborn scars, continue with the applications of topical Kaloidon Gel after consulting with the treating physician. In general, the results may vary from person to person and from scar to scar.

CONTRAINDICATIONS

The use of this patch is contraindicated in cases of known or suspected allergies to any of the ingredients contained in the medical device.

PRECAUTIONS

Do not apply to open wounds or skin ulcers. Use with clean hands. Do not apply in combination with other topical products.

Do not reuse a used patch, always apply a fresh patch every day. Zip the bag closed after removing a patch.

Store in a cool, dry place, away from direct light and heat sources and at a temperature no greater than 30° C.

Keep out of the sight and reach of children.

Do not use after the expiration date on the package. For external use only. Do not discard the container after use.

The product has no known adverse effects, however, it is not recommended for use if there are known sensitivities to one or more ingredients.

If you experience any side effects, stop treatment immediately and consult your doctor or pharmacist.

 LFM S.r.l. Via Monterosso 273
21042 Caronno Pertusella (VA) - www.lfm.it



Do not reuse



See the instructions for use



Do not use if the packaging is not intact



Keep away from light



30°C
Maximum temperature 30°C



LOT Lot number



Use by date



FOR EXTERNAL USE ONLY



CE 0477

Date of last revision of the operating instructions: July 2025

500000771-A1